



Nano Coating for Neurolight Treatment Lenses

We trust you are happy with the Neurolight devices you are using in your clinic.

All versions are pre-coated with a specially formulated nano-material that is both hydro-phobic and oleo-phobic. This class of coatings is widely used in medical environments.

In a clinical environment, with normal cleaning procedures, the device coating is good for about a month and is easily re-applied. It is not necessary to re-apply this coating to units used exclusively at home by patients.

If you do not use a protective barrier such as a clear plastic wrap or bag between the device and the patient in your clinical environment, we strongly recommend you re-apply the coating on a specific day of the month every month, to all units in the clinic.

The procedure is very simple. Spray a small amount on the lens, gently wipe in using a microfiber cloth, let sit for an hour or until dry and then LIGHTLY buff it out with the cloth. Each press of the pump delivers about 5ml. The bottom section requires only 1-2 pumps, each side only 1 pump per application.

The coating is available in 12oz and 4oz bottles. To order, simply call 888 426-8794 or email us at info@nltus.com.

Regards,

The entire team at Neurolight Treatment...where patient results are paramount.